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Pelvimeter graduated in centimeters and inches for interior measurements between symphysis and promontory

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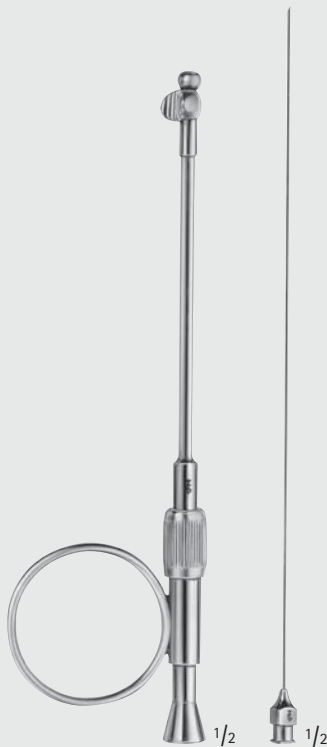
When the guide sheath is introduced at the spot to be injected the needle (mounted on the syringe) is passed through the guide sheath and the tissue is finally injected. Injuries of the vagina are thus avoided. Depth of puncture is limited to 10 mm. Thumb ring as well as a small support for middle- or forefinger facilitate the handling. Thumb ring slides on the guide sheath and can be fixed by a knurled nut to any size of the hand. The finger support can be turned round on the guide sheath so left-handers can use it, too. Completely away from the range of actions is the finger support when it is moved towards the thumb ring, e.g. for anesthesia of the pudendus.

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Special set for transvaginal injection
for paracervical anesthesia as well
as anesthesia of the pudendus

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Obstetrical stethoscopes see section AC

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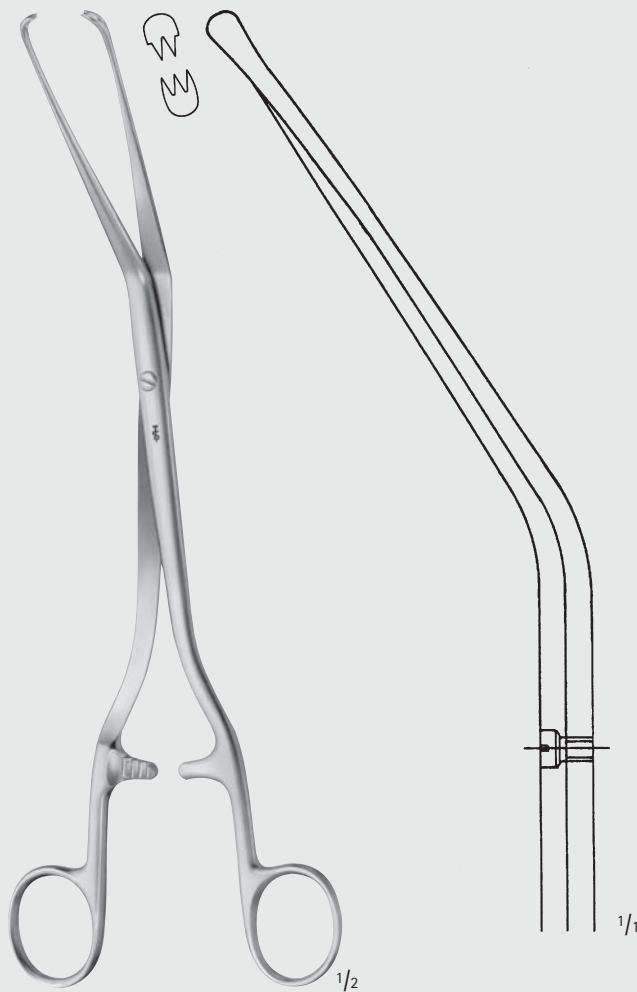
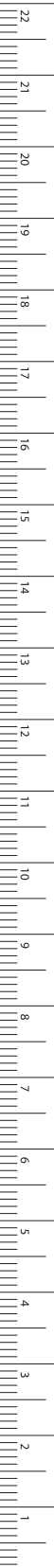
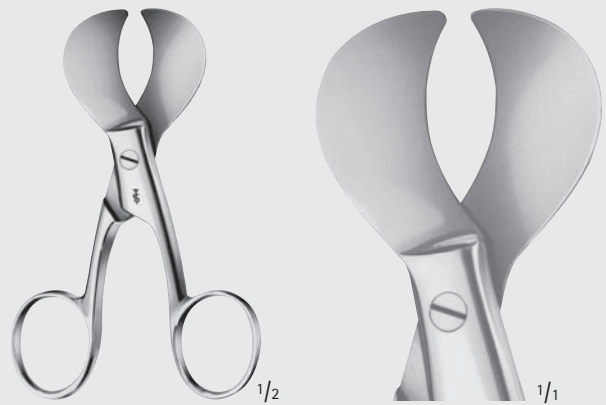
Nabelschnurschere
 Umbilical cord scissors
 155 mm, 6"

ET152R

Nabelschnurschere
 Umbilical cord scissors
 105 mm, 4 1/8"

BC796R

BC797R

BC796R	135 mm	5 1/4"
BC797R	160 mm	6 1/4"

Nabelschnurscheren
 Umbilical cord scissors

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Dammscheren, Nabelschnurdilatator, Nabelschnurklemme

Episiotomy Scissors, Umbilical Dilator, Umbilical Cord Clamp

BRAUN-STADLER

ET140R 145 mm 5 3/4"

ET141R 200 mm 8"

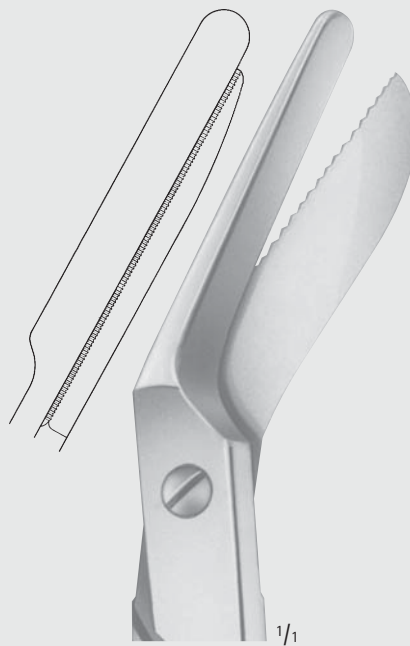
Dammscheren

Episiotomy scissors

ET140R

1/1

ET141R

1/1

ET142R

Dammschere
Episiotomy scissors
240 mm, 9 1/2"

1/1

1/1

ET130R

Nabelschnurdilatator
Umbilical dilator
95 mm, 3 3/4"

1/2

COLLIN

ET160R

Nabelschnurklemme
Umbilical cord clamp
85 mm, 3 3/8"

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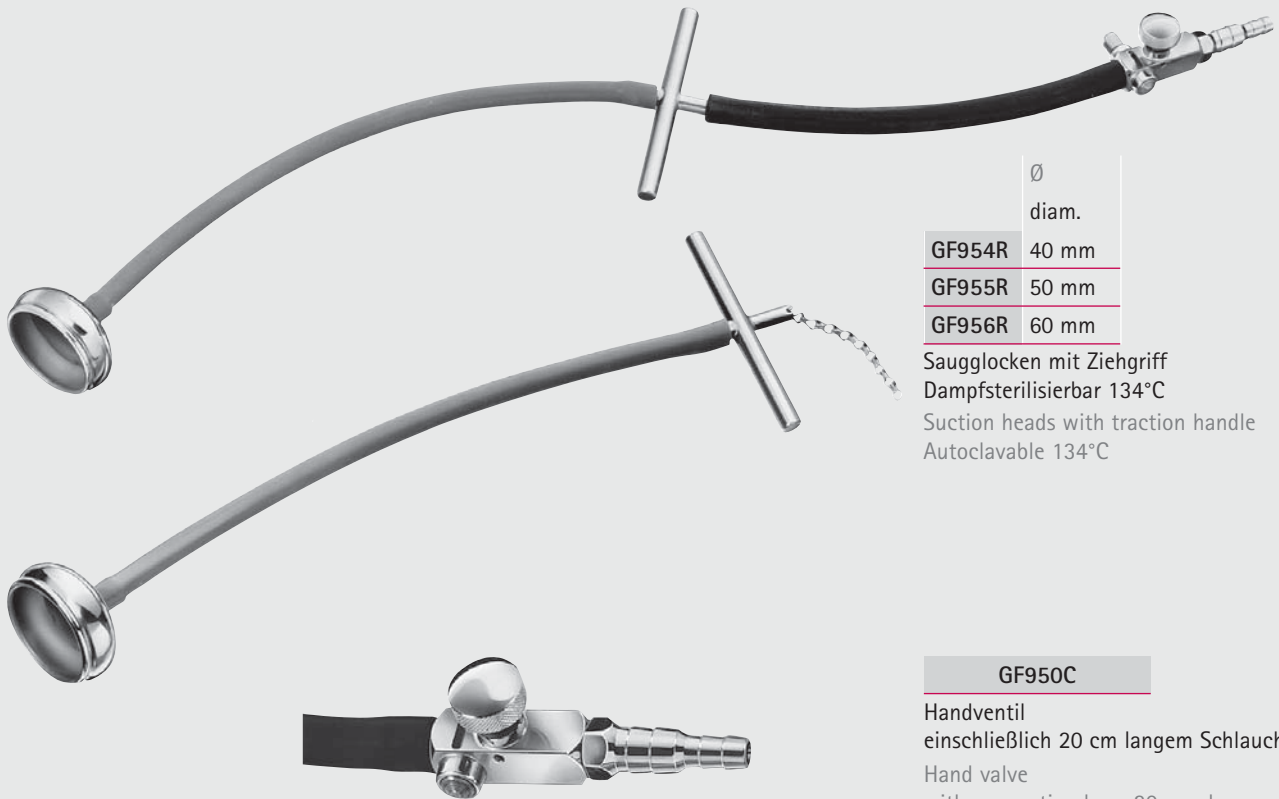
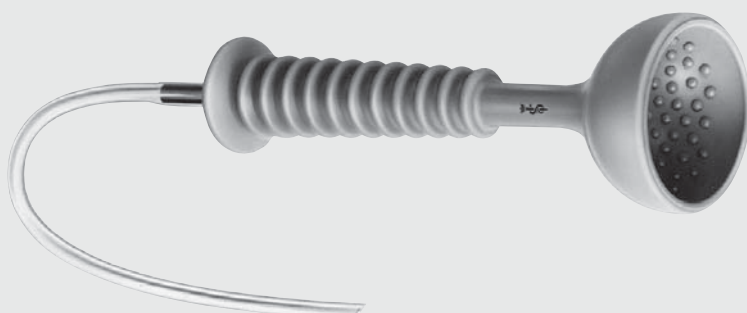
With the hand valve GF950C Aesculap electric suction pumps can also be used for Malmström vacuum extraction. The vacuum hand valve GF950C can also be connected to central suction system.

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Function: The vacuum required can be constantly preset by the adjusting screw. This vacuum exists then up to the closed quick-shutter, thus the vacuum in the suction head is pressed from the left to the right, the mounted suction head is immediately put under the preset vacuum. In case the preset vacuum has to be changed, this can be done with the adjusting screw. For immediate removal of the suction head simply close the quick-shutter, thus the vacuum in the suction head is eliminated at once whereas the vacuum in the suction jar remains unchanged.

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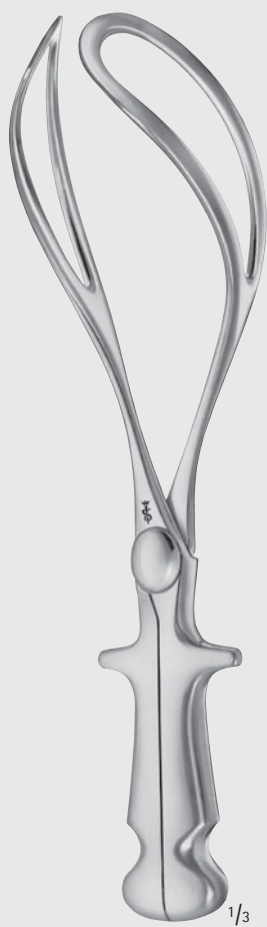
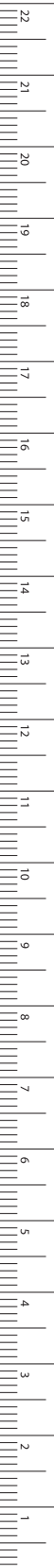
MILLIMETERS

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Geburtslöffel, Uterus-Klemmzange

Obstetrics Lever, Caesarean Section Hemostatic Forceps

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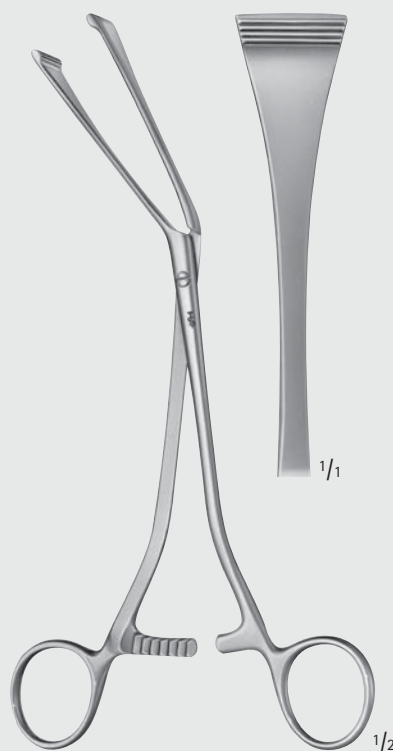
Caesarean section hemostatic
forceps, angled,
for edges of the transverse
incision in the lower segment

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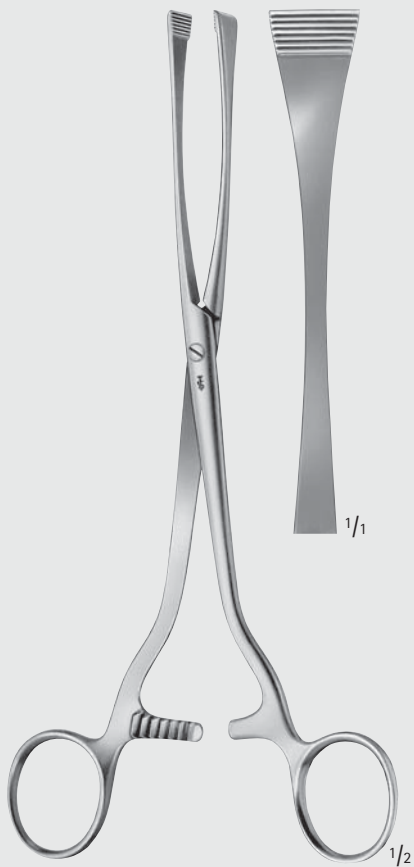
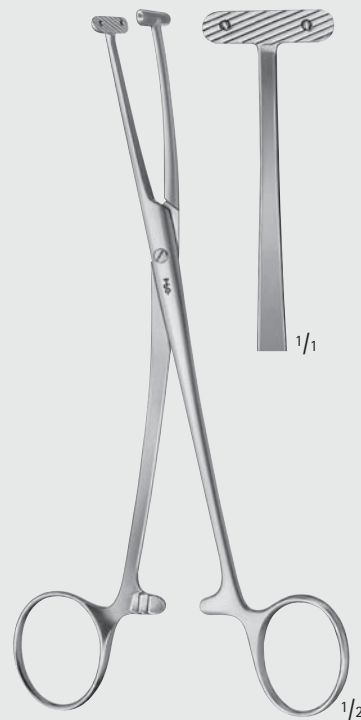
Caesarean section haemostatic forceps, for edges of the transverse incision in the lower segment

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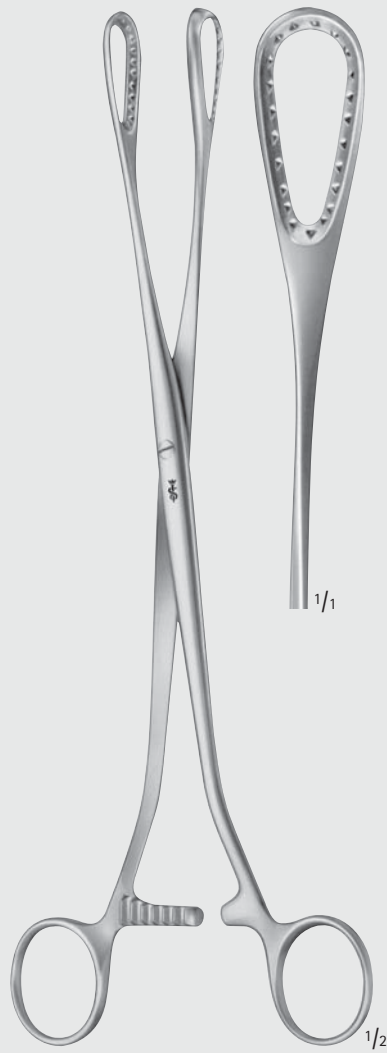
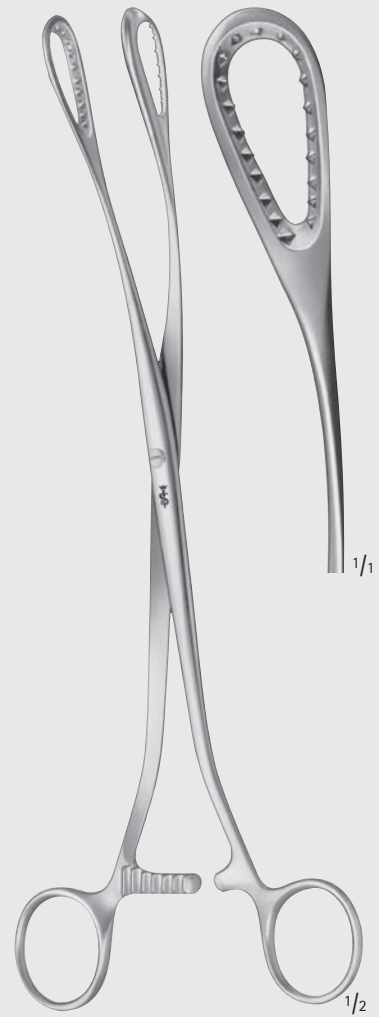
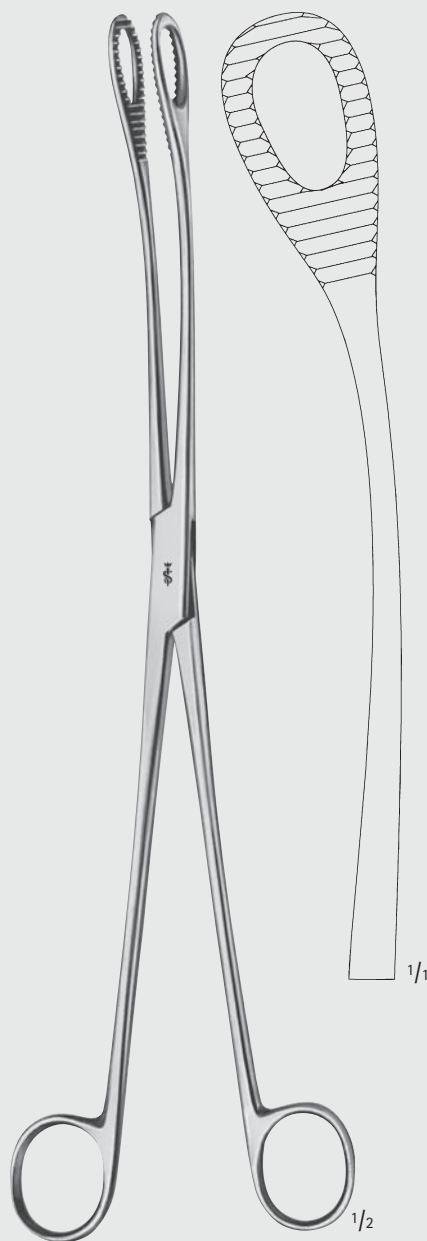
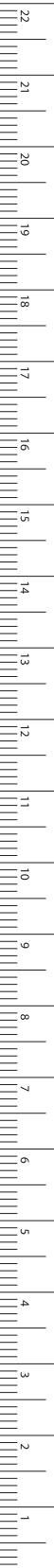
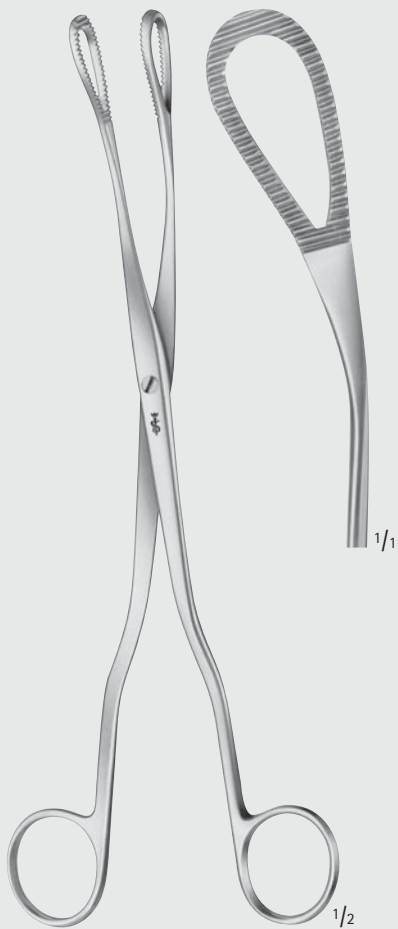
240 mm, 9 1/2"

FF010R

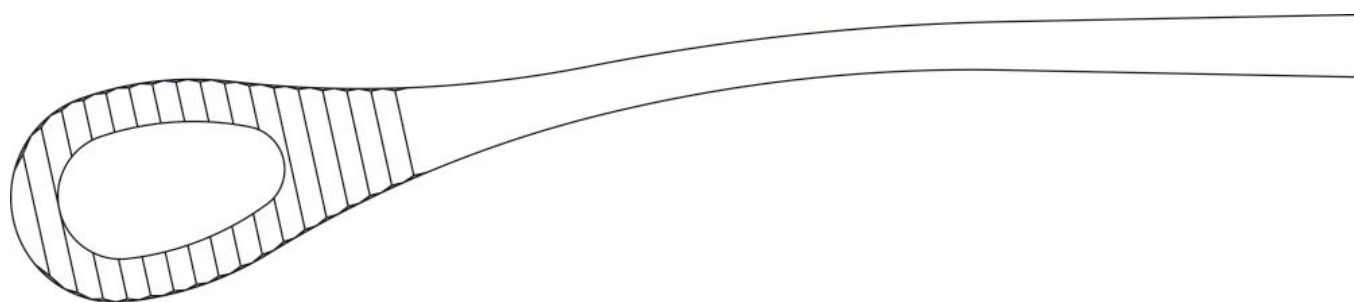
DEMARTEL Scalp Flap Clamp, 135 mm (5 1/4"), serrated, non-sterile, reusable

Product search type	Scalp Flap Clamp
Author	DEMARTEL
Aesculap Product	
Groups	General Surgical Instruments
Length	135.00 mm
Length (Inch)	5 1/4
Jaw profile	serrated
Discipline	Neurosurgery
Surgical Step	Diss.-Tissue-Others
Data Matrix	Yes
Product contains	
Latex	No
Sterility	non-sterile
Disposable /	
Reusable	reusable

BF136R

KELLY Sponge Forceps, curved, 320 mm (12 1/2"), serrated, fenestrated, box lock, with ratchet, non-sterile, reusable

Product search type	Sponge Forceps
Author	KELLY
Aesculap Product	
Groups	General Surgical Instruments
Altern.	
Productname	Polyp forceps, Organ grasping Forceps
Geometry	curved
Length	320,00 mm
Length (Inch)	12 1/2
Blades / Jaw	fenestrated
Jaw profile	serrated
Instrument joint	box lock
Ratchet	with ratchet
Discipline	General Surgery, Gynaecology, Hand / Foot Surgery, Cardiac Surgery, Maxillo-Facial Surgery, Neurosurgery, Orthopaedics, Pediatric Surgery, Plastic Surgery, Spine Surgery, Thoracic Surgery, Traumatology, Urology, Vascular Surgery, Ear-Nose-Throat Surgery
Surgical Step	Disinfection
Type of Sterilization	not sterilized ex works
Data Matrix	Yes
Product contains	
Latex	No
Sterility	non-sterile
Disposable /	
Reusable	reusable

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